


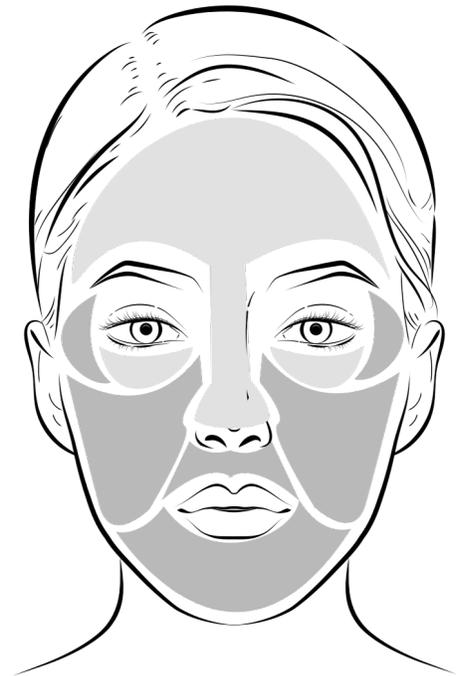


Depth / Stroke Chart


Recommended **depth settings**

	Depth 0.0 - 0.3mm
	Depth 0.5 - 3.0mm



Recommended **stroke patterns**

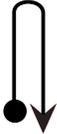	Start treatments at the top of the forehead (0.0mm depth) – with no pressure – toward the mid-face region, with vertical strokes moving horizontally outward. Use linear, slightly overlapping strokes at a rate of about 1 inch per second for even coverage.
	Keep contact with skin when changing direction from a vertical to horizontal motion and apply pressure based on treatment, patient-tolerability and desired results. Ensure skin is kept taut using the hand free from device and use plenty of glide before and during treatment.
	Small circular motions are recommended for optimal coverage.

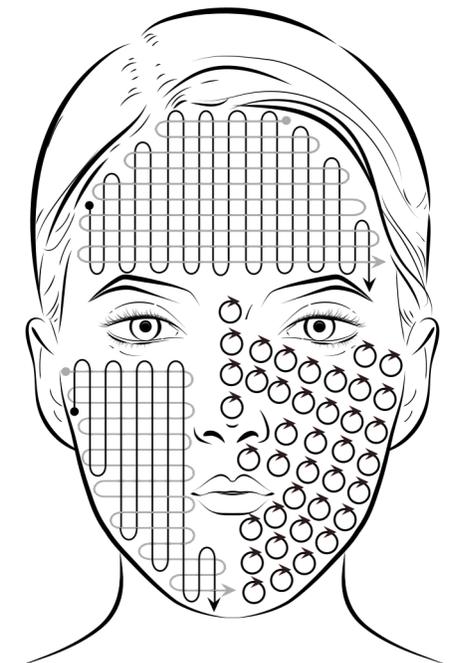

STD 0300-12 Rev. C